

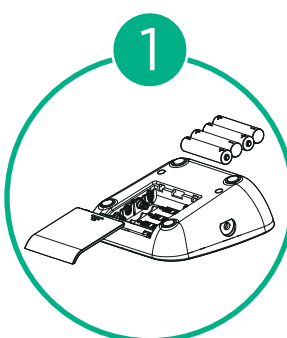

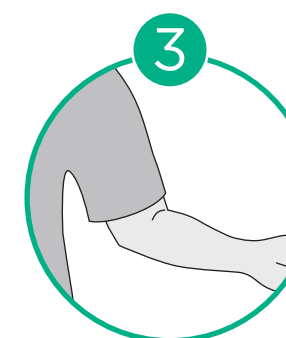




Kinetik TMB-1970 UK version Quick Start Guide
Size: 50(W) x 75(H) mm [Cover and Back Cover Size: 54(W) x 79(H) mm]
Date: 20220425

Colour : (Green) Pantone 339C (CMYK reference is C84 M0 Y59 K0) ■
(Black)

Kinetik Logo Colour : 
Pantone 382C  **kinetik**
WELLBEING
(CMYK reference is C30 M0 Y100 K0)
Pantone Cool Grey 11 
(CMYK reference is C44 M34 Y22 K77)

50mm

75mm

- 
Unpack Blood Pressure Monitor and insert batteries.
- 
Plug the connector from cuff into the monitor on the left side.
- 
Remove any thick clothing from your arm.
- 
Sit in a relaxed position for 5 minutes.
- 
Place your arm resting on a firm surface.
- 
Wrap the cuff on the left arm and keep level with your heart.
- 
Press the START button to begin measurement.

For further help contact us at

Email
customercare@kinetikwellbeing.com



Live Chat
kinetikwellbeing.com

Phone
+44 (0) 1483 937967

Kinetik TMB-1970 UK QS Guide 20220425

54mm

79mm

In association with  

Quick Start Guide

TMB-1970

Fully Automatic Blood Pressure Monitor

Cover

54mm

79mm



for more information visit :
kinetikwellbeing.com

Back Cover